PUSH CARE KNEE BRACE

INTRODUCTION

Push care products provide effective solutions when your freedom of movement is restricted by an injury or disorder. The products have been approved by medical specialists and scientists.

Your new knee brace is a Push care product, characterised by a high level of effectiveness, comfort and quality. The Push care knee brace provides pain relief and a sense of additional stability by means of adjustable compression.

Push care products are innovative and are manufactured for you with great care. This ensures maximum security and carefree use.

Read this product information carefully before you use the Push care knee brace.

If you have any questions, please consult your reseller. Alternatively, ask your question through info@push.eu.

PRODUCT CHARACTERISTICS

The Push care knee brace provides compression around the knee joint, improving the sense of stability of the joint.

The brace supports the patella on both sides. The compression and patella support can be adjusted as needed. The horseshoe-shaped patella element stabilises and relieves the patellofemoral joint. The open construction of the brace's upper part allows the brace to be fitted easily. A proper anatomic fit and silicone lining contribute to the brace remaining in position during walking.

The brace is made of Sympress™, a high-quality comfortable material. The microfibre used makes the inside soft to the touch and keeps the skin dry. The Push care knee brace is easy to apply and can be washed without any problem.

INDICATIONS

- Osteoarthritis of the knee joint
- Patellofemoral osteoarthritis
- Rheumatoid complaints
- Residual functional instability
- Retropatellar pain complaints
- After meniscus operations
- In case of capsular irritation

There are no known disorders where the brace cannot or should not be worn. In the case of poor blood circulation in the leg, a brace may cause hyperaemia. Make sure not to apply the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

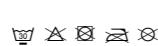
FITTING / SETTING

The knee brace is available in 5 sizes. Position the upper leg at an angle of 90° in relation to the lower leg. Then measure the circumference at the narrowest section underneath the knee joint. The table on the packaging shows the correct brace size for your knee circumference.

Loosen all Velcro fasteners to facilitate fitting.

- Put the brace over the knee joint, until the preformed patella element encloses the knee. Secure the brace by closing the Velcro fasteners **1** and **2**.
- Close the straps **3** and **4** diagonally across the upper part. Adjust to the most comfortable pressure, by tightening or loosening the straps.

CARE AND MAINTENANCE

Before washing, close the Velcro to protect the brace and other laundry.

The brace can be washed at temperatures up to 30°C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used; after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

To extend the brace's service life, it should be dried after use, if necessary, before being stored.

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Optimal functioning can only be achieved by selecting the right size. Therefore, fit the brace before purchasing, to check its size. The expert staff at your reseller's is able to explain the best way of fitting the Push brace.
- We recommend that you keep this product information in a safe place, to be able to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Each time before use, check your Push brace for signs of wear or ageing of parts or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.

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The brace supports the patella on both sides. The compression and patella support can be adjusted as needed. The horseshoe-shaped patella element stabilises and relieves the patellofemoral joint. The open construction of the brace’s upper part allows the brace to be fitted easily. A proper anatomic fit and silicone lining contribute to the brace remaining in position during walking.

The brace is made of Sympress™, a high-quality comfortable material.

The microfibre used makes the inside soft to the touch and keeps the skin dry. The Push care knee brace is easy to apply and can be washed without any problem.

INDICATIONS

- Osteoarthritis of the knee joint
- Patellofemoral osteoarthritis
- Rheumatoid complaints

- Residual functional instability
- Retropatellar pain complaints
- After meniscus operations
- In case of capsular irritation

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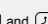
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
A. Pull the brace over the knee joint, until the preformed patella element encloses the knee. Secure the brace by closing the Velcro fasteners  and .

B. Close the straps  and  diagonally across the upper part. Adjust to the most comfortable pressure, by tightening or loosening the straps.

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A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

To extend the brace’s service life, it should be dried after use, if necessary, before being stored.

IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Optimal functioning can only be achieved by selecting the right size. Therefore, fit the brace before purchasing, to check its size. The expert staff at your reseller’s is able to explain the best way of fitting the Push brace.
- We recommend that you keep this product information in a safe place, to be able to consult it for fitting instructions whenever needed.
- Consult a brace specialist if the product requires individual adaptation.
- Each time before use, check your Push brace for signs of wear or ageing of parts or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.

Push® is a registered trademark of Nea International by

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