PUSH MED BACK BRACE

INTRODUCTION

Push med products provide effective solutions in the case of a disorder or reduced functioning of the locomotor apparatus. Approved by physicians and scientists.

Your new back brace is a Push med product, characterised by a high level of effectiveness, comfort and quality. The Push med back brace helps to relieve pain by supporting the lumbar vertebral column. This increases physical strength. Mechanical support, adjustable pressure and good position retention constitute the basis of its performance.

Push med products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push med back brace. If you have any questions, please consult your reseller or send an e-mail to info@push.eu.

PRODUCT CHARACTERISTICS

Your Push med back brace ensures a proper position of the vertebral column by means of metal bracing elements adjusted to the anatomy of your back. The brace supports the back from the sacral vertebrae to the lower thoracic vertebrae (S2 to Th 8). Tensioning the elastic bands provides compression and increases the support offered by the stiffeners.

Because of its design and the materials used, the brace is comfortable to wear during standing or sitting activities, both at home and at work. Its anatomical fit is suitable for both men and women, and guarantees good position retention. The close body-fit of the brace renders it invisible underneath your clothes. Compression can be adjusted easily to suit your preferences. The brace is made of Sympress™, a high-quality comfortable material. The use of microfibres makes the back part feel soft and keeps the skin dry. Microperforations allow the outer material to breathe. The Push med back brace is easy to apply and can be washed without any problem.

INSTRUCTIONS FOR INDIVIDUAL FITTING BY A PROFESSIONAL SPECIALIST

The specialist determines whether the shape of the stiffeners needs to be adjusted for you. The shape of the metal stiffeners can be adjusted to the individual user by bending with force.

INDICATIONS

- Spondylolysis
- Spondylolisthesis
- Degenerative disorders of the lumbar vertebral column (lumbar arthrosis)
- Discopathy
- Osteoporotic lumbar impression fractures
- Postsurgical treatment

There are no known disorders where the brace cannot or should not be worn. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

DETERMINING SIZE / FITTING

The back brace is available in 6 sizes. The size is determined by the circumference of the waist. The table on the packaging shows the correct brace size for your waist.

- Wear the back brace against the skin or over your undergarments. Please ensure that the brace is positioned in the centre of the vertebral column at the level of the lower back. The bottom of the brace covers the hips. The shape of the bracing elements must fit your back when standing up with a straight back.
- Loosen the elastic bands before fitting the brace. Hold the brace by the lower edge and close the Velcro or use the grips. Tension and close the elastic bands. If the pressure experienced is too tight, the brace should be slackened. If the brace has been fitted properly, you should feel a comfortable, straight position. You may also use the grips to close the brace.

WASHING INSTRUCTIONS

Before washing, close the Velcro to protect the brace and other laundry. Before washing, open the zip and remove the metal bracing elements from the brace.

The brace can be washed at temperatures up to 30 °C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

To extend the brace's service life, it should be dried after use, if necessary, under being stored.

IMPORTANT

- Für die richtige Anwendung der Push brace may require diagnosis by a doctor or therapist.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before use, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. Save the product information, to be able to reread the fitting instructions later.
- Consult a specialist if the product requires individual adaptation.
- Check your Push brace each time before using for signs of wear or ageing of parts or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.

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