PUSH ORTHO ANKLE ORTHESIS AEQUI

INTRODUCTION

Push Ortho products provide effective solutions in the case of a disorder or reduced functioning of the locomotor apparatus. Approved by physicians and scientists.

Your new ankle orthosis is a Push ortho product, characterised by a high level of effectiveness, comfort and quality. The Push Aequi increases ankle stability and helps to control pain. This increases physical strength. Mechanical support, adjustable pressure and good position retention constitute the basis of its performance.

Push ortho products are innovative and are manufactured for you with great care. This ensures maximum security and more freedom of movement. To promote satisfied use.

Read the product information carefully before you use the Push Aequi. If you have any questions, please consult your reseller or send an e-mail to info@push-brace.com.

PRODUCT CHARACTERISTICS

The Push Aequi is an effective ankle orthosis, which strikes a balance between efficient mechanical support of an unstable ankle and good wearer comfort. The rigid support is situated exclusively on the medial (inner) side of the ankle and has a cavity for the malleolus. The lateral side is made from preformed foam, which easily adapts to the individual anatomy. The bracing material is constructed in such a way that it is very rigid in the centre, becoming softer towards the edges to ensure a good fit and wearer comfort. The orthosis is fastened by means of a diagonal strap. The elastic straps apply pressure around the ankle. The Push Aequi can be used in the functional treatment of ankle injuries and as a long term support in cases of chronic ankle instability. The inversion/eversion movement (inward / outward movement of the foot) is restricted while plantar/dorsal flexion (bending and stretching) is allowed, for adequate freedom. This means that no inhibition is felt during normal walking and running. The numbers on the straps make fitting the orthosis simplicity itself! The Push Aequi is worn over the sock, and optimal stabilisation can only be achieved if it is worn with a closed (lace-up) shoe.

INDICATIONS

- Serious (residual) instability (under high stress)
- Treatment of acute lateral ankle ligament lesions
- Follow-up treatment of conservative or operated ankle fractures.
- Arthrosis with instability without accumulation of fluid
- Secondary prevention of ankle ligament lesions (under high stress without accumulation of fluid)

There are no known disorders where the orthosis cannot or should not be worn. In the case of poor blood circulation in the limbs, a orthosis may cause hyperaemia. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

FITTING

- The Velcro fastenings are marked with symbols. These are to help you fasten the Velcro in the correct sequence. First ☐ on ☐, then ☐ on ☐ etc.
- The Push Aequi is worn over the sock
 - Place the orthosis around the foot. The rigid part is on the inside of the foot. Put the long straps to the sides
 - Fasten the diagonal strap marked with ☐ (fig. 1)
 - Take hold of the elastic strap marked with ☐ at the outside of the foot. Pull this diagonally across the instep, wrap it around the ankle just above the heel and fasten in place (fig. 2)
 - Finally take hold of the elastic strap on the inside of the foot, marked with ☐. Pull this from the instep across the dorsum (top of the foot), wrap it around the ankle over the first strap and fasten in place (fig. 3). The instep is the lower part of the foot, where the strap starts.

WASHING INSTRUCTIONS

Before washing, close the Velcro to protect the brace and other laundry.

The brace can be washed at temperatures up to 30°C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

To extend the brace's service life, it should be dried after use, if necessary, before being stored.

IMPORTANT

The efficient use of Push Aequi ankle orthosis depends on the correct diagnosis of the reasons for pain or instability. Always check first with your doctor, a physiotherapist or another specialist before using a orthosis.

If this orthosis is to function effectively it is important:

- to follow the washing instructions
- that the elastic material can still be stretched
- the Velcro fastens efficiently
- the orthosis is not severely damaged (no tears, loose seams)
- the orthosis is not subjected to temperatures higher than 70°C.
- please be aware that an ill-fitting brace does not function properly.

Push® is a registered trademark of Nea International bv.
P.O. Box 1558, 6201 BN Maastricht, Netherlands
www.push-brace.com

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