PUSH ORTHO ANKLE BRACE AEQUI JUNIOR

Please read this product information carefully before using the Push Aequi Junior. Aandachtig durch.

INTRODUCTION

Push products provide effective solutions for a variety of disorders of the locomotor apparatus. The products can also be used as a preventive measure after previous injuries of the joints.

The Push ortho ankle brace Aequi Junior is a Push Braces product from the so-called ortho category. Push ortho products are innovative and are carefully manufactured. They aim to provide the highest level of comfort, maximum support and effective protection. The Aequi Junior ankle brace was especially designed for children with unstable ankles.

The Aequi Junior is characterised by a high degree of support, effectiveness, comfort and quality. The product guarantees enhanced stability and helps to control pain. A major advantage of wearing this brace is that it increases load bearing capacity. This restores the ankle joint's full functionality more quickly, and in particular more safely.

PRODUCT CHARACTERISTICS

The Aequi Junior is an effective ankle brace for children (or adults with a small instep) aged between 6 and 12, with an instep of 23-27 cm (shoe sizes 30-37). This ankle brace is suitable for both the left and right ankle and is available in a single size.

The brace offers a great balance between effective mechanical ankle support and optimal wearer comfort. It consists of a sturdy foot shell and a set of elastic and non-elastic straps. The Aequi Junior provides the required support of the ligaments. Elastic straps also provide pressure around the ankle joint.

The inversion-eversion movement (inward-outward movement of the foot) is restricted, while plantar-dorsal flexion (bending and stretching) is allowed, for adequate freedom. This means that no inhibition is felt during normal walking and running.

The numbers on the straps allow for an easy fitting of the brace. Tests have shown that children are capable of fitting the brace themselves. For a proper functioning of the brace, however, it is important that parents or carers initially observe and provide any help needed and inspect whether the Aequi Junior has been fitted correctly.

Maximum stabilisation is achieved when the brace is worn inside a closed (lace-up) shoe. When children play indoors, the Aequi Junior can also be worn without a shoe, while retaining the function of stabilisation.

INDICATIONS

- In the case of early functional therapy for injuries to the medial or lateral ankle ligaments
- Follow-up treatment of conservative or operated ankle fractures
- In the case of residual or other instability (increased tendency to sprain)
- To prevent (recurrence of) ankle ligament injuries

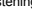
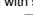
There are no known disorders where the brace cannot or should not be worn.

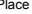
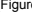
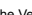
Note: In the case of poor blood circulation in the limbs or swelling, a brace may cause hyperaemia. Avoid applying the brace too tightly in the beginning.

Please note any warning signs: If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

Always follow the advice of your doctor or therapist.

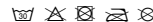
FITTING

The Velcro fastenings are marked with symbols. These are to help you fasten the Velcro in the correct sequence. First , then , then  on  etc.

- The Push Aequi Junior is worn over the sock.
- Place the brace around the foot. Lay the long straps to the sides.
- Fix the diagonal straps  and  and the shell closely around the ankle (Figure A).
- Wrap the long elastic strap  diagonally across the instep and fasten it on the Velcro .
- Wrap the long elastic strap  diagonally across the instep and fasten it on the Velcro .

If the brace is worn by a child, always check whether it has been fitted properly. If the product should be left on during the night, it should be fitted less tightly.

WASHING INSTRUCTIONS

The brace can be washed at temperatures up to 30 °C on a delicate fabrics cycle. Do not use any bleaching agents.

Hang the brace up to dry in the open air (not near a heating source or in a tumble drier).

To extend the brace's service life, it should be dried after use, if necessary, before being stored.

IMPORTANT

Proper use of the Aequi Junior requires a correct diagnosis of the causes of pain or complaints. Always consult your GP, a physiotherapist or a specialist before using the brace.

If this brace is to function effectively it is important:

- to follow the washing instructions
- that the elastic material can still be stretched
- the Velcro fastens efficiently
- the brace is not severely damaged (no tears, loose seams)
- the brace is not subjected to temperatures higher than 70 °C.

ANY QUESTIONS?

For any questions concerning the product, please consult the reseller who provided the brace, or send an e-mail to info@push.eu.

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