## PUSH CARE BACK BRACE

### INTRODUCTION

Push care products provide effective solutions when your freedom of movement is restricted by an injury or disorder. Approved by physicians and scientists.

Your new back brace is a Push Care product, characterised by a high level of effectiveness, comfort and quality. The Push care back brace guarantees pain relief by supporting the lumbar vertebral column.

Push care products are innovative and are manufactured for you with great care. This ensures maximum security and carefree use.

Read the product information carefully before you use the Push care back brace. If you have any questions, please consult your reseller or send an e-mail to [info@push.eu](mailto:info@push.eu).

### PRODUCT CHARACTERISTICS

Your Push care back brace offer relief for lower-back complaints by evenly distributed compression. A soft, comfortable pad applies additional pressure locally on the lower back. This pad can be moved.

Because of its design and the materials used, the brace is comfortable to wear during standing or sitting activities, both at home and at work. Its anatomical fit is suitable for both men and women, and guarantees good position retention.

The close body-fit of the brace renders it invisible underneath your clothes. Compression can be adjusted easily to suit your preferences. The brace is made of Sympress™, a high-quality comfortable material. The use of microfibras makes the back part feel soft and keeps the skin dry. The Push care back brace is easy to apply and can be washed without any problem.

### INDICATIONS

- Chronic non-specific lower-back complaints
- For back-straining activities (secondary prevention)
- Pseudoradicular complaints (sciatica)

There are no known disorders where the brace cannot or should not be worn. Avoid applying the brace too tightly in the beginning.

Pain is always a sign of warning. If the pain persists or the complaint deteriorates, it is advisable to consult your doctor or therapist.

### DETERMINING SIZE / FITTING

The back brace is available in 6 sizes. The size is determined by the circumference of the waist. The table on the packaging shows the correct brace size for your waist.

- ❶ Wear the back brace against the skin or over your undergarments. Please ensure that the brace is positioned in the centre of the vertebral column at the level of the lower back. The bottom of the brace covers the hips.
- ❷ Hold the brace at the bottom and close the Velcro strip. In doing so, tension the elastic material. After fitting, the brace should apply a comfortable pressure. If the pressure experienced is too tight, the brace should be slackened. You may also use the grips to close the brace.

### WASHING INSTRUCTIONS

Before washing, close the Velcro to protect the brace and other laundry.

The brace can be washed at temperatures up to 30 °C, on a delicate fabrics cycle or by hand. Do not use any bleaching agents.

A gentle spin-dry cycle can be used: after this, hang the brace up to dry in the open air (**not** near a heating source or in a tumble drier). The pad needs to be replaced with the point facing up ❸.

To extend the brace's service life, it should be dried after use, if necessary, before being stored.

### IMPORTANT

- Proper use of the Push brace may require diagnosis by a doctor or therapist.
- Optimal functioning can only be achieved by selecting the right size. Fit the brace before use, to check its size. Your reseller's professional staff will explain the best way to fit your Push brace. Save the product information, to be able to reread the fitting instructions later.
- Consult a specialist if the product requires individual adaptation.
- Check your Push brace each time before using for signs of wear or ageing of parts or seams. Optimal performance of your Push brace can only be guaranteed if the product is fully intact.

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